

# Fibronectin Aggrecan Complex Test (FACT™) Joint Diagnostic Sample Collection Kit

## Indications

The Fibronectin Aggrecan Complex Test (FACT<sup>™</sup>) is an enzyme linked immunosorbent sandwich assay (ELISA) that measures the presence of a Fibronectin-Aggrecan Complex (FAC) and other related biomarkers in a fluid specimen taken from patients with spine or joint related pain. FAC is a unique molecular complex that has been reported in peer-reviewed publications to be associated with painful inflammation.

FACT can be used in the assessment of joint or spine related pain in patients greater than 18 years of age who have an acute onset of pain (or an acute exacerbation of baseline pain). A physician may utilize FACT as an adjunct to existing clinical and radiologic information to determine the source of a patient's pain. A fluid specimen can be obtained from the patient prior to treatment, intraoperatively / intraprocedurally upon obtaining access to the disc or joint, or after treatment during a follow up visit. The use of FACT has not yet been evaluated for the assessment of inflammatory arthritides, infectious arthritis, crystalline arthropathies, or autoimmune diseases.

FACT is NOT intended as the sole basis for treatment or disease monitoring decisions. All other available clinical information should be taken into consideration when counseling the patient regarding the treatment of pain. FACT is NOT intended to screen for or predict the outcome of treatments. Caution: Federal law restricts this device to sale by or on the order of a physician.

This Laboratory Developed Test was developed and its performance characteristics determined by Cytonics Corporation. Cytonics is regulated under the Clinical Laboratories Improvement Amendments of 1988 and is qualified to perform high-complexity clinical testing (CLIA #10D2007773). FACT has not been cleared or approved by the U.S. Food and Drug Administration.

# Instructions For Use: Joint Diagnostic Sample Collection Kit

The Joint Diagnostic Collection Kit includes two collection tubes, each containing a light sensitive protease inhibitor, in a black collection tube pouch together with a Test Request Form. The patient name and date of birth must be included on the collection tubes, on the black collection tube pouch, and on the Test Request Form for sample tracking and patient identification purposes.

Prior to the Procedure: Complete all fields on the Test Request Form and get both the patient's and the physician's signatures.

#### Specimen Collection Procedure: Joint Aspiration / Lavage

- 1. Premark syringe(s) for the aspiration or lavage of each suspected diseased joint
- 2. Engage the needle into the patient's joint capsule fluoroscopy or ultrasound guidance may be used to confirm needle placement
- 3. Aspirate the joint by drawing fluid back into the syringe
- 4. NOTE: If the aspiration of the joint results in less than 0.35cc of fluid, an additional sterile syringe containing 10cc to 12cc of
  - physiologic saline should be used to lavage the joint space
    - a. Attach the syringe with saline to the needle placed in the patient's joint
    - b. Inject the saline into the joint capsule and aspirate using an in / out pumping action to draw fluid back into the syringe
    - c. The return of saline should take on a slightly cloudy or hazy appearance
  - Detach the syringe from the needle

5.

- 6. Transfer the fluid from the syringe into the collection tube that is pre-marked for the aspirated / lavaged collection site
- 7. Repeat the procedure if additional sites are to be aspirated or lavaged
- 8. Notate the specimen collection method used (Saline Lavage or Synovial Fluid Aspiration) on the Test Request Form
- 9. Notate the site from which the specimen was collected on the Test Request Form

## **Specimen Packaging and Return Shipment**

- 1 Place collection tubes into the black zip-lock bag and close
- 2 Insert bag into silver bubble pouch and seal with flap tape
- 3 Put the pouch in the Cytonics box with the fully completed Test Request Form and close
- 4. Place the box into the clear FedEx Clinical Pak and seal with flap tape
- 5. Call FedEx for pick up: 800-463-3339. Leave at reception desk for pick up
- 6. Samples must be shipped within 24 hours of collection

Cytonics Corporation 2013. Form 01LSC010-003 CLIA Number: 10D2007773 For more information, please visit <u>www.Cytonics.com</u>